




ORagene-DNA

REF OG-500 and OG-510

REF OG-600 and OG-610

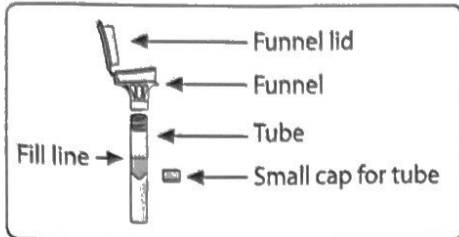

Collection precautions:

7 time

Do NOT eat, drink, smoke or chew gum for 30 minutes before giving your saliva sample.

Do NOT remove the plastic film from the funnel lid.

Intended use: For the collection of human DNA from saliva samples.

Contents: Kit contains stabilizing liquid.

Warnings and precautions: Wash with water if stabilizing liquid comes in contact with eyes or skin. Do NOT ingest. See MSDS at www.dnagenotek.com.

Small cap may pose a choking hazard.

Storage: 15°C / 30°C

Summary and explanation of the kit:

Oragene-DNA is a self-collection kit that provides the materials and instructions for collecting and stabilizing saliva specimens.

Label legend:


- Consult package insert
- Collect saliva by (Use by)
- In vitro diagnostic medical device
- Catalog number
- CE Marking
- Caution, consult instructions for use
- Storage instructions
- Authorized Representative
- Manufacturer
- Lot number

USER INSTRUCTIONS


Read all instructions prior to collection

Procedure:


Most people take between 2 and 5 minutes to deliver a saliva sample following steps 1 to 5.

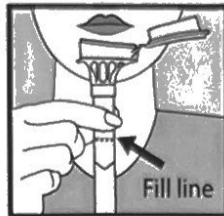


- 1 Spit into funnel until the amount of liquid saliva (not bubbles) reaches the fill line shown in picture #1.

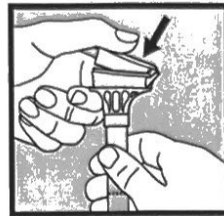


- 2 Hold the tube upright with one hand. Close the funnel lid with the other hand (as shown) by firmly pushing the lid until you hear a loud click. The liquid in the lid will be released into the tube to mix with the saliva. Make sure that the lid is closed tightly.

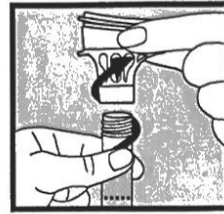

- 3 Hold the tube upright. Unscrew the funnel from the tube.

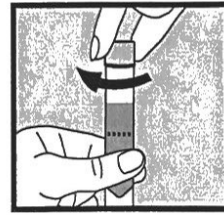

- 4 Use the small cap to close the tube tightly.

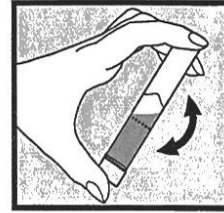

- 5 Shake the capped tube for 5 seconds. Discard or recycle the funnel.

CE

For In Vitro Diagnostic Use

DNAGENOTEK



Made in Canada
DNA Genotek Inc.
3000 - 500 Palladium Drive
Ottawa, ON, Canada K2V 1C2

Superior samples
Proven performance

Toll-free (North America): 1.866.813.6354
Tel.: +1.613.723.5757 • Fax: +1.613.723.5057
info@dnagenotek.com
www.dnagenotek.com

EC REP

Emergo Europe, Prinsessegracht 20, 2514 AP The Hague, The Netherlands

Australian Sponsor: Emergo Australia, Level 20, Tower II, Darling Park, 201 Sussex Street, Sydney, NSW 2000 Australia

Oragene-DNA is not available for sale in the United States.

*Oragene is a registered trademark of DNA Genotek Inc.

Some DNA Genotek products may not be available in all geographic regions, contact your sales representative for details. All DNA Genotek protocols, white papers and application notes, are available in the support section of our website at www.dnagenotek.com.

Patent (www.dnagenotek.com/legalnotices)

© 2017 DNA Genotek Inc., a subsidiary of OraSure Technologies, Inc., all rights reserved.

PD-PR-118 Issue 5/2017-02