





ORACollect

for pediatrics


REF OC-175

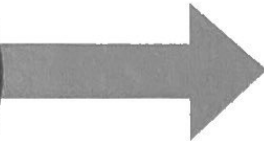

Instructions for sample collection:

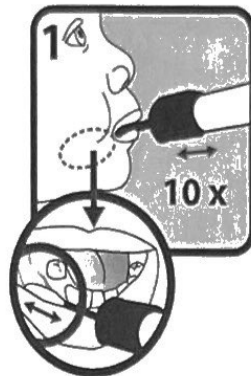


Open package and remove collector without touching sponge tip. Place sponge as far back in the mouth as comfortable and rub along the lower gums (see close up image) in a back and forth motion. Gently rub the gums 10 times. If possible, avoid rubbing the teeth.

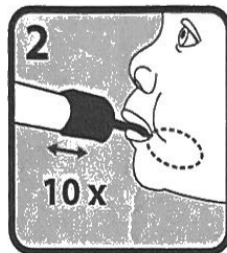

Gently repeat rubbing motion on the opposite side of the mouth along the lower gums for an additional 10 times.



Hold the tube upright to prevent the liquid inside the tube from spilling. Unscrew the blue cap from the collection tube without touching the sponge.

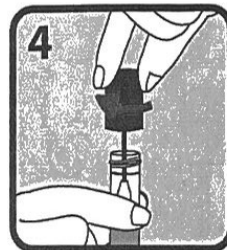

Turn the cap upside down, insert the sponge into the tube and close cap tightly.



Invert the capped tube and shake vigorously 10 times.

Collection precautions:

Ensure the sponge tip does NOT come into contact with any surface prior to collection.

Donor should NOT eat, drink, smoke or chew gum for 30 minutes before collecting oral sample. *1 time*

Intended for the collection of human DNA from pediatric oral samples.

Contents: Contains 1 collection kit.

Warning and precautions: Sponge tip is a choking hazard. Caution should be used when inserting sponge into the mouth. For assisted collections do NOT leave infant or child unattended.

Wash with water if liquid comes in contact with eyes or skin. Do NOT ingest. See MSDS at www.dnagenotek.com

Storage: 15/25°C

Summary and explanation of the kit:

ORACollect for pediatrics is an assisted collection kit that provides the materials and instructions for collecting and stabilizing human DNA from pediatric oral samples.

Label legend:

- Manufacturer
- Catalog number
- Non-sterile
- Lot number
- Collect sample by (Use by)
- Storage instructions

For Research Use Only

Not for use in diagnostic procedures

DNAGENOTEK



*Superior samples
Proven performance*

DNA Genotek Inc.
3000 - 500 Palladium Drive
Ottawa, ON, Canada K2V 1C2

Toll-free (North America): 1.866.813.6354
Tel: +1.613.723.5757 • Fax: +1.613.723.5057
info@dnagenotek.com
www.dnagenotek.com

*ORACollect is a registered trademark of DNA Genotek Inc.

Some DNA Genotek products may not be available in all geographic regions, contact your sales representative for details. All DNA Genotek protocols, white papers and application notes, are available in the support section of our website at www.dnagenotek.com.

Patent (www.dnagenotek.com/legalnotices)
© 2019 DNA Genotek Inc., a subsidiary of OraSure Technologies, Inc., all rights reserved.
PD-PR-00542 Issue 2/2019-10